Reusable Mask



Step 1 Cut two main cotton fabric pieces.

Cut two flannel lining pieces.

Step 2 Sew the curved seams together ¼" seams through out pattern. Snip the curve.

Turn to right side, push seam to one side and topstitch the curve.

Step 3 Fold over ¼" of the little side parts. Stitch down close to the raw edge.

Step 4 Place right sides together and pin the top of mask pieces together. Stitch 1/4". Repeat with the bottom.

Turn right side out + press

Step 5 Finish little side pieces by leaving ½' open at the beginning and

the end.

penings 1/2" openings 7


This will make an opening to insert the elastic piece into the side part.

Tie with a non-slip knot. I tie right over left and then left over right! Adjust loop of elastic to fit around an ear. Leave the knot out so the person who wears the mask can adjust it needed.

Step 6 Topstitch the top and bottom of the mask.

The hospital will wash all masks before use.

Y you need 2 pieces of elastic, 10" to 12" long can use 14". 1/2" on round cord clastic Make a template with template plastic or card board

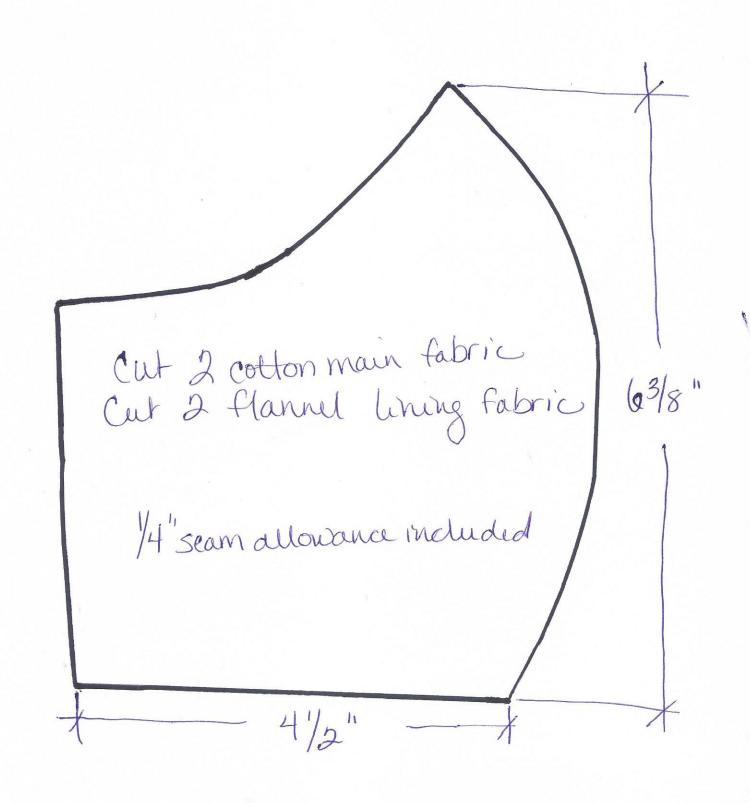

Reusable Face Mask - Sky Lakes Medical Center

Step 1: Cut two main cotton fabric pieces and cut two flannel lining pieces from pattern


Step 2: With rs together, sew a 1/4 inch seam on the curved seam of the two main






Snip the curves. Press the seam to one side. Topstitch the seam on the main

cotton pieces. Do the same with the flannel pieces.

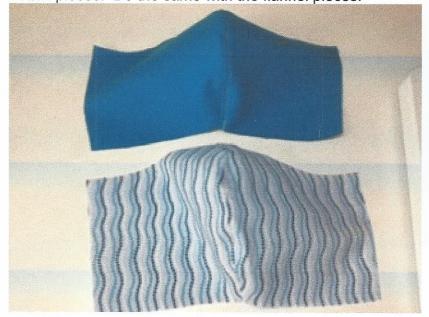


Step 3: On the sides of the pattern pieces (the area that will be by the ears) fold under 1/4 to the wrong side of the fabric and topstitch. Do this to both the main pattern piece

and the flannel piece.

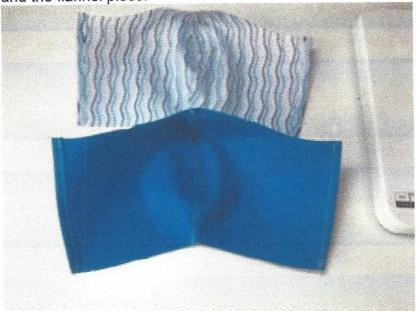

Step. 4: Place right sides of main piece and flannel piece together and stitch at 1/4 seam allowance. Do this to the top and bottom.

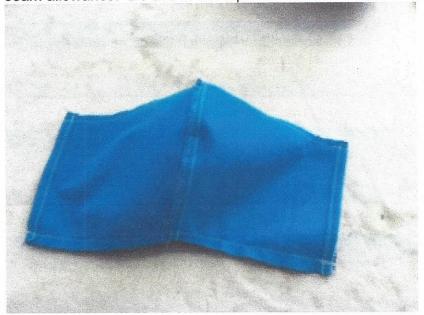


Turn to right side and press.



Step. 5: At each side, leave 1/2 opening on the top and bottom so you can insert elastic through. Stitch the sides together leaving that $\frac{1}{2}$ opening on top and bottom.



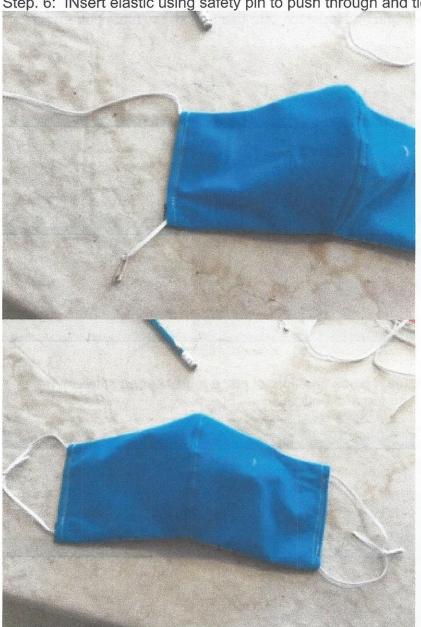

Step. 6: INsert elastic using safety pin to push through and tie ends.

Step 7: Topstitch top and bottom.

